

Parameter settings for 3shape DentalSystem™

Recommendations

	ZENOTEC SCI.ENCE CoCr	ZENOTEC SCI.ENCE IPS e.max inlays	ZENOTEC SCI.ENCE IPS e.max crowns	ZENOTEC CAD/CAST (comparable to ZENOTEC PMMA)	ZENOTEC SCI.ENCE abutments
Cement gap	0.045	0.01	0	0.01	X
Extra cement gap	0.05	0.09	0.09	0.04	X
Distance to prep.	1.2	0.5	0.5	1	X
Smooth distance	0.2	0.2	0.2	0.2	X
Drill radius	0.6	0.6	0.6	0.6	X
Drill compensation offset	0.5	0.5	1.5	1.5	X
Wall thickness	0.4	0.45	0.5	0.4	X
Wall height	1	–	1	1	X
Margin line offset	0.1	–	0.25	0.1	X
Offset angle	65	–	70	65	X
Extension offset	0.1	–	0.1	0.1	X
Output format	STL	STL	STL	STL	STL